



885 E. Collins Blvd., Suite 110
Richardson, Texas 75081 USA
(972) 783-7005 Tel
(800) 787-0378 Toll Free Tel
(972) 235-3446 Fax
(844) 272-6624 Toll Free Fax

SUTURE STAY

Suture Stay	Description
RTC-1020S	FASTRACT Suture Stays (Pair)
RTC-1030S	FASTRACT Suture Stay (Single)

© 2012 Chase Medical and the Chase Medical logo are registered service marks of Chase Medical.

All Rights Reserved
Printed in USA

900296 REV-13

STERILE	EO
---------	----

English
SUTURE STAY / 1 UNIT

DESCRIPTION: The Suture Stay is a tightly coiled spring that is tapered to ensure that it will lock into the suture stay slot of the Sternal Retractor.

WARNINGS:

- Read instructions carefully. The attending clinician is solely responsible for the setup and use of this device.
- Observe aseptic technique when placing suture stays in the retractor arm.
- Contents of unopened, undamaged package are sterile.
- Do not resterilize. Single use only.
- Dispose of this device according to hospital procedure for contaminated material.

CAUTIONS:

- Federal (USA) law restricts this device to sale by or on the order of a physician.
- Observe all additional warnings and/or cautions contained in the Directions for Use below.

Directions for Use:

1. Slide a suture stay firmly into the suture stay holder on the Chest Retractor arm with the tab on the end of the spring facing up.
2. When fully inserted rotate the spring so that the tab locks into the oval on the retractor arm.
3. Insert sutures individually into the suture stay holder; do not attempt to place multiple sutures simultaneously.

LIMITED WARRANTY AND DISCLAIMER OF LIABILITY

Chase Medical products ("Product") are manufactured under carefully controlled conditions and Chase Medical warrants the Product will perform in accordance with its labeling, and will be free from defects in material and workmanship under normal use for one year from the date of shipment.

Chase Medical's sole liability shall be limited to replacement of, or credit for, any Product returned to Chase Medical and which Chase Medical determines, in its sole discretion, to have been defective in either material or workmanship at the time of shipment.

Chase Medical has no control over the conditions under which the product is handled or used by third parties, and specifically assumes no liability with respect to instruments reused, reprocessed or resterilized, in contraindication to the warnings in this document . THEREFORE, CHASE MEDICAL DISCLAIMS ALL WARRANTIES, BOTH EXPRESS AND IMPLIED, WITH RESPECT TO THE PRODUCT, INCLUDING BUT NOT LIMITED TO ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. CHASE MEDICAL SHALL NOT BE LIABLE TO ANY PERSON OR ENTITY FOR ANY MEDICAL EXPENSES OR DAMAGES CAUSED BY ANY USE, DEFECT, FAILURE OR MALFUNCTION OF THE PRODUCT, WHETHER A CLAIM FOR SUCH DAMAGES IS BASED UPON WARRANTY, CONTRACT, TORT OR OTHERWISE. NO PERSON HAS ANY AUTHORITY TO BIND CHASE MEDICAL TO ANY REPRESENTATION OR WARRANTY WITH RESPECT TO THE PRODUCT.

The exclusions and limitations above are not intended to and should not be construed as intending to contravene mandatory provisions of applicable law. If any part of this Limited Warranty and Disclaimer of Liability is held to be unenforceable or in conflict with applicable law by a court of competent jurisdiction, the validity of the remaining portions shall be enforced as if this Limited Warranty and Disclaimer of Liability did not contain the part held invalid.